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FANTESTIC® APPLIKATOR SYNCRO

**Hersteller**

R-dental Dentalerzeugnisse GmbH - Hamburg  
 Telefon +49-(0)40-22757617 - Fax +49-(0)40-22757618 - www.r-dental.com

**FANTESTIC® APPLIKATOR SYNCRO****Bestell-Nr.: FFK3405**

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**FANTESTIC® APPLICATOR SYNCRO****Order-No.: FFK3405**

The FANTESTIC® APPLICATOR SYNCRO is an applicator for commercial composite tips.

**Important precautions to reuse**

The FANTESTIC® APPLICATOR SYNCRO must be sterilized before the first use at the patient and after every use at the patient with steam at 273 °F (for 3 min, air steam disinfectant, keep attention on instructions for use of manufacturer).

The FANTESTIC® APPLICATOR SYNCRO can further on be disinfected with alcohol and commercial disinfectants, which contain 1,5 % glutaraldehyd. But the treatment may last not longer than 60 min.

The FANTESTIC® APPLICATOR SYNCRO can not be disinfected with other disinfection methods; in these cases there is no warranty.

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for defections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2013 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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FANTESTIC ist ein eingetragenes Warenzeichen der R-dental Dentalerzeugnisse GmbH.
© 2013 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
R-dental Dentalerzeugnisse GmbH - Hamburg - Germany