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FANTESTIC® FILL

CE 0482

Hersteller

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 040 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

FANTESTIC® FILL

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REF FFT34XX (in Tips, je nach Farbe) ☒

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FANTESTIC® FILL

REF FFS33XX (in syringes, depends on shade)

REF FFT34XX (in tips, depends on shade) ②

Description and indication

FANTESTIC® FILL is a light-cure and well mouldable micro hybrid composite with high abrasion stability for direct anterior - and posterior tooth fillings in the scope of an adhesive filling treatment (classes I-V).

Characteristics and composition

FANTESTIC® FILL is radio-opaque, highly polishable and shows a natural aesthetic with an excellent colour stability. It is based on BIS-GMA-resin and inorganic filler particles of a size of 0.05-1.5 µm. The total filler load is 81 wt.-% and 65 vol.-%.

The shades of FANTESTIC® FILL products correspond to VITA®-shades.

FANTESTIC® FILL meets the requirements of DIN EN ISO 4049.

Recommended procedures**1. Preparation**

Prior to the preparation a cleaning of the tooth with a fluoride-free paste (e. g. fluor of pumice with water) is recommended.

2. Shade matching

Ascertain the tooth shade while teeth are still moist and select the appropriate FANTESTIC® FILL shade (shades corresponds to VITA®-shades).

3. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

4. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules before filling treatment, cleaned and if necessary (Cp, P) the pulp should be protected with calcium hydroxide, Cal de Luxe®, Ca(OH)₂, ("ready-to-use" in a portion blister, REF CDL2503).

Then apply the Total-etch-technique (Etch & Rinse) according to section 5 a. and 5. b. or the Self-etch-technique according to section 6.

5 a. Total-etch-technique (Etch & Rinse) according to the instructions for use

Apply FANTESTIC® ETCH (REF EGS3000) precisely first on enamel and then on dentin for 15 s (reaction time enamel at least 15 s, dentin 15 s). For teeth of 1. dentition or enamel with high fluoride content let FANTESTIC® ETCH effect longer. Then rinse for about 20 s with water and dry the enamel with oil- and water-free air. The dentin should be maintained in a visibly moist condition. The etched enamel should appear chalky white. If this is not the case, repeat the etching procedure of the enamel using a shorter reaction time.

After use waste the disposable application cannula and screw on the original cap to reseal the syringe.

5 b. Application of the one-component adhesive FANTESTIC® FLOWSIVE or FANTESTIC® FLOWSIVE 2 according to the instructions for use

FANTESTIC® FLOWSIVE (REF FFF3100) or - FLOWSIVE 2 (REF FFF3105) will be applied using a MICROBRUSH (REF MPB3981) onto the etched dentin and enamel areas for about 30 s. Aerate FANTESTIC® FLOWSIVE or - FLOWSIVE 2 for about 15 s in a weak air stream and light-cure it for about 20 s. Repeat this step at least one time for reaching a higher adhesive strength.

6. Application of the self-etch one-component adhesive FANTESTIC® UNIVERSAL BOND (Self-etch-technique) according to the instructions for use

FANTESTIC® UNIVERSAL BOND (REF FUB6000) can be used as a self-etch one-component adhesive for the self-etch-technique, for selective enamel etching and for the total-etch-technique (Etch & Rinse).

7. Application of the flowable composite FANTESTIC® FLOWFILL according to the instructions for use

FANTESTIC® FLOWFILL (REF FFS32XX, XX depends on the shade) can be used as a lining material for an optimal adaptation of the mouldable composite FANTESTIC® FILL within the cavity. At small cavities FANTESTIC® FILL can be applied directly.

8. Application of the composite FANTESTIC® FILL

FANTESTIC® FILL should then be applied in thin layers of max. 2 mm into the cavity. Placing with a non-metallic instrument is recommended.

9. Light-cure

FANTESTIC® FLOWFILL and FANTESTIC® FILL can be light-cured with commercial polymerization units (halogen or LED). Each layer must be cured separately. The spectra of the light source should be around 475 nm.

When using a polymerization lamp with an intensity of $\geq 1,000$ mW/cm² cure each layer of lighter shades (e. g. A1, A2, C2) **20 s** and darker shades (e. g. A3,5, B3) **30 s**.

10. Finishing and high gloss polishing

After removing the matrix FANTESTIC® FILL can be contoured with finishing diamonds, flexible discs or carbide burs, and can be polished with polishing discs or polishing brushes to high gloss.

Side effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material as pulp capping (Cp, P), see section 4.

Precautions to application, product life and/or storage

Avoid contact of the etching gel FANTESTIC® ETCH and calcium hydroxide Cal de Luxe® with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

Do not use FANTESTIC® FILL in case of allergic reactions. Avoid contact of FANTESTIC® FILL with soft tissues, skin or eyes. Medical gloves do not protect against methacrylates.

Do not adjust the viscosity of the composite by using any resin. Do not store FANTESTIC® FILL in proximity of eugenol containing materials nor let it come in contact with eugenol containing materials. Eugenol can impair hardening and cause discoloration.

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F! Avoid exposure to sunlight. Close syringe immediately after use to avoid exposure to light. Keep out of the reach of children!

*VITA is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2019 R-dental Dentalerzeugnisse GmbH. All rights reserved.