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FANTESTIC® OxyBlok

**Hersteller**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
 Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

FANTESTIC® OxyBlok
Bestell-Nr.: FOS3970

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FANTESTIC® OxyBlok
Order-No.: FOS3970

Description and indication

FANTESTIC® OxyBlok is a protecting-gel. The protecting-gel inhibits the contact of oxygen on a composite layer. As a result the so called oxygen inhibition layer will be reduced or inhibited.

Because of the application of FANTESTIC® OxyBlok composite surfaces of light-cure or self-curing that is dual-cure composites can be fully polymerized.

Recommended procedures**1 a. Application of FANTESTIC® OxyBlok - filling treatment**

After an adhesive filling treatment the protecting-gel is applied in a thin layer of about 1-2 mm onto the last composite layer. Then the composite layer will be light-cured or self-cured. After this the gel will be rinsed intensively with water.

The composite surface can then be polished to high-gloss smearless finish.

1 b. Application of FANTESTIC® OxyBlok - luting treatment

After a luting treatment of restorations the protecting-gel is applied in a thin layer of about 1-2 mm onto the cementing seam. Then the composite will be light-cured or self-cured. After this the gel will be rinsed intensively with water.

The composite seam can then be worked out smearless.

Precautions to application, product life and/or storage

Expiration date see label. Do not use after expiration date.

Do not store above 77 °F! Avoid exposure to sunlight. Keep out of the reach of children!

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for deflections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2016 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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FANTESTIC ist ein eingetragenes Warenzeichen der R-dental Dentalerzeugnisse GmbH.
© 2016 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE