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FANTESTIC® Universal Bond

CE 0482

Hersteller:R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com**FANTESTIC® UniversalBond**

REF FUB6000

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REF FUB6000

Description

FANTESTIC® UniversalBond is a universal, self-etching, light-cure one-component adhesive for the Self-Etch Technique, the Selective Enamel Etching and the Total-Etch Technique (Etch & Rinse). It is suitable for moist, wet and dry surfaces. Based on innovative MDP technology FANTESTIC® UniversalBond allows an excellent adhesion.

Note: FANTESTIC® UniversalBond can only be used with self-curing and dual-cure composites if a sufficient light-cure is guaranteed. Otherwise the self-etching and dual-cure two-component adhesive system FANTESTIC® FLOWSIVE SE (REF FFF5155) must be used.

Indications

FANTESTIC® UniversalBond is indicated in the context of Self-Etch-Technique, Selective Enamel Etching and Total-Etch-Technique (Etch & Rinse)

- for direct restorations with light-cure composite and compomers,
- for core build-ups with light-, self- and dual-cure core build-up composites, e.g. FANTESTIC® CORE DC, FANTESTIC® Z CORE DC, if light-cure of the adhesive is possible,
- for luting of inlays, onlays, crowns and bridges with light-, self- and dual-cure composite cements e. g. FANTESTIC® FLOWFILLCEM DC and if light cure of the adhesive is possible and
- for repair of fractured composite restorations.

Side effects

In rare cases FANTESTIC® UniversalBond may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, FANTESTIC® UniversalBond should not be used. Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of calcium hydroxide or a bioactive pulp capping material, see section 2.

Incompatibilities

Do not use FANTESTIC® UniversalBond in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store FANTESTIC® UniversalBond in proximity of eugenol containing materials. Unpolymerized FANTESTIC® UniversalBond should not come in contact with eugenol containing materials.

Recommended procedures

1. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

2 Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction. If possible, margins should have a slight (0.5-1.0 mm) bevel placed in the enamel to increase the surface area for higher bond strength.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide (Cal de Luxe®, (Ca(OH)₂, "ready-to-use" in a portion blister, REF CDL2503) according to the instructions for use or with a bioactive pulp capping material, like ReViCal (REF RVC1002) according to the instructions for use.

3. Optional conditioning with an etching gel

Optional FANTESTIC® ETCH (REF EGS3000) can be used for the Selective Enamel Etching or the Total-Etch Technique (Etch & Rinse) to improve the adhesive strength to enamel.

Note: Unprepared enamel surfaces must always be conditioned with FANTESTIC® ETCH.

Precaution at/after etching: It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as described above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

3 a. Selective Enamel Etching

The etching gel FANTESTIC® ETCH will be applied precisely on the enamel for at least 15 s on enamel. For teeth of 1. dentition or enamel with high fluoride content let FANTESTIC® ETCH effect correspondingly longer. Then rinse for about 20 s with water and dry the enamel with oil- and water-free air. The dentin must not dry out. The etched enamel must appear chalky white. If this is not the case, the enamel should be etched again with a shorter exposure time.

3. b. Total-Etch Technique (Etch & Rinse)

The etching gel FANTESTIC® ETCH will be applied precisely beginning with the enamel bevels onto enamel and dentin and conditioned for at least 15 s on enamel and 15 s on dentin. For teeth of 1. dentition or enamel with high fluoride content let FANTESTIC® ETCH effect correspondingly longer. Then rinse for about 20 s with water and dry the enamel with oil- and water-free air. The dentin must not dry out. The etched enamel must appear chalky white. If this is not the case, the enamel should be etched again with a shorter exposure time.

4. Application of the adhesive FANTESTIC® UniversalBond

FANTESTIC® UniversalBond will be applied using a MICROBRUSH (REF MPB3981) onto the physiologically wet dentin and enamel areas and agitate intensively for 20 s (applying the adhesive on the tooth surface without agitation is insufficient). It must be ensured that the material is distributed homogeneously over the entire surface. Then aerate the solvent in FANTESTIC® UniversalBond for 5-10 s in a weak air stream until a glossy and uniform layer results. Then light-cure the adhesive for 20 s.

FANTESTIC® UniversalBond can be light-cured with commercial polymerization units with a light intensity > 500 mW/cm² (Halogen) or > 1000 mW/cm² (LED) with a wavelength of 400-500 nm. The ambient light of the dental lamp may start polymerization of the adhesive.

Then a light-cure, self-curing or dual-cure composite can be applied immediately.

5. Application of a composite

For an optimal result a thin layer of a light-cure flowable composite (e. g. FANTESTIC® FLOWFILL or tZeen® FLOW) should be applied and light-cured. Then a light-cure, self-curing or dual-cure composite can be applied according to the corresponding user instructions.

6. Storage

Do not store above 77 °F! Avoid exposure to sunlight.

Precautions to application, product life and/or storage

Avoid contact of calcium hydroxide Cal de Luxe® and the pulp capping material ReViCal with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Do not use FANTESTIC® Universal Bond in case of allergic reactions. Avoid contact of FANTESTIC® Universal Bond with soft tissues, skin or eyes. In case of contact with skin flush immediately with water and soap. In case of contact with eyes, rinse immediately with copious amounts of water and if necessary consult a physician. Commercial medical gloves do not protect against a sensitizing effect of methacrylates.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Close the bottle immediately after use.

Expiry date see label. Do not use after expiry date.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants that this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH assumes no other liability and does not grant any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for the application and the appropriate use of the product according to its intended purpose. If any defects occur to the product within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal situation, including warranty, contract, negligence or intent.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.