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Deutsch - MOK1050XX-01/24

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R-SI-LINE® MONO S

CE 0482

### Hersteller:

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • E info@r-dental.com • I www.r-dental.com

REF MOK1050 (R-SI-LINE® MONO S)

REF MOK1050FS (R-SI-LINE® MONO S FS (fast set))

### Produktbeschreibung

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**REF MOK1050FS (R-SI-LINE® MONO S FS (fast set))**

### Product description

R-SI-LINE® MONO S (- FS) in auto-mixing cartridges is a super hydrophilic, medium viscosity, addition curing one-phase (monophase) precision impression material on vinylpolysiloxane base (A-silicone). It can be extruded directly onto the teeth or onto the tray.

R-SI-LINE® MONO S (- FS) is very hydrophilic and thus improves the wettability and shows an excellent detail reproduction especially of subgingival areas and other moist surfaces.

R-SI-LINE® MONO S (- FS) is very thixotropic and firm (S). It stays in the position on the tray but under pressure it flows.

R-SI-LINE® MONO S (- FS) meets the requirements of DIN EN ISO 4823 Type 2 (medium-bodied consistency).

### Indications/intended use

R-SI-LINE® MONO S (- FS) is indicated for

- one-phase (monophase) impressions for crowns, bridges, inlays, partial and complete dentures and
- relining impressions, functional impressions and impressions for prosthetic restorations.

### Performance features

The performance features of the product meet the requirements of the intended use.

### Contraindications

Not known.

### Patient target group

Persons who are treated during a dental procedure.

### Intended users

This medical device should only be used by a professionally trained dental practitioner.

### Application

#### 1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE MIXING CANNULA. The mixing pistol is ready for application.

For direct fine application of the R-SI-LINE® MONO S (- FS) the R-SI-LINE® INTRA-ORAL-TIPS (REF IOT2000) should be used.

#### 2. Impression taking

Before application a small amount of material (size of a pea) should be pressed out and should be wasted immediately.

Load the auto-mixed R-SI-LINE® MONO S (- FS) onto the R-SI-LINE® SILICONE ADHESIVE (REF SIA1070) coated tray and/or onto the teeth or the prepared areas.

Insert the tray into the mouth not later than 2 min (the fast set variant not later than 1 min 30 s) after extruding (working time at 23 °C (74 °F)).

Maintain the position of R-SI-LINE® MONO S for at least 3 min (the fast set variant R-SI-LINE® MONO S FS at least 2 min) in the mouth.

Note: The filled MIXING CANNULA functions as a seal until the next application. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting.

#### 3. Casting

The impression can be poured 30 min after removal from the mouth. There are no other time limitations.

#### 4. Electroplating

All impressions can be copper- or silverplated.

#### 5. Disinfection

The impression can be disinfected with disinfectant solutions especially for materials based on addition-curing silicones. Please follow the manufacturer's instructions for use.

#### 6. Storage

Do not store above 25 °C (77 °F)! Avoid exposure to sunlight. Do not use after expiry date.

### Additional notes

Do not use R-SI-LINE® MONO S (- FS) with standard condensation reaction type silicone impression materials.

Do not use latex or sulfur-containing polymeric gloves because they may interfere with the setting reaction of the impression material.

Keep out of the reach of children!

### Disposal

Disposal of the product according to local authority regulations.

### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

### Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

### Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2024 R-dental Dentalmaterialien GmbH. All rights reserved.