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ARROW TRAXX®

CE 0482

**Hersteller:**

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

**ARROW TRAXX®**

REF ATR2150

**Produktbeschreibung**

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### ARROW TRAXX® REF ATR2150

#### Product description

ARROW TRAXX® is a light-cure model-composite suitable for extension of impression trays, for placement of position stops on impression trays or for modelling of occlusion rims.

ARROW TRAXX® is ready for use and easy to handle. It can be used within sufficient time to mould under day and room light. It can be light-cured with commercial polymerization units (halogen or LED or commercial light boxes) with a wave length of 400-500 nm and with a light intensity of at least 1,000 mW/cm<sup>2</sup>.

#### Indications/Intended use

ARROW TRAXX® is indicated for

- extension of impression trays,
- placement of position stops on impression trays and
- modelling of occlusion rims.

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

Not known.

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Incompatibilities with other materials

Do not use ARROW TRAXX® in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store ARROW TRAXX® in proximity of eugenol containing materials. Unpolymerized ARROW TRAXX® should not come in contact with eugenol containing materials.

#### Application

##### 1. Application

Take the required quantity of ARROW TRAXX® and model it for the intended application within the working time of about 2 min (working time at 74 °F) and then positioned according to the indication (extension of impression trays, placement of position stops on impression trays and modelling of occlusion rims).

The polymerization time is about 20 s for layers of maximum 2 mm (depending on the light intensity of the polymerization unit).

Close the jar immediately after use to avoid exposure to light!

##### 2. Storage

Do not store above 25 °C (77 °F)! Protect from direct sunlight. Do not use after expiry date.

#### Additional notes/warnings

The ambient light of the dental lamp may start polymerization of the composite.

Do not adjust the viscosity of the composite restorative material by using any resin.

Avoid contact with skin, mucous membranes and eyes.

Unpolymerized ARROW TRAXX® may have an irritant effect and can lead to sensitization against methacrylates.

Commercial medical gloves do not protect against methacrylates.

Keep out of the reach of children!

#### Disposal

Disposal of the product according to local authority regulations.

#### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

ARROW TRAXX® is a registered trademark of R-dental Dentalerzeugnisse GmbH.

#### Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

#### Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2024 R-dental Dentalmaterialien GmbH. All rights reserved.