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FANTESTIC® ETCH

CE 0482

Hersteller:
R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

FANTESTIC® ETCH REF EGS3000

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FANTESTIC® ETCH
REF EGS3000

Description

FANTESTIC® ETCH is an etching gel which contains 37 % phosphoric acid with an excellent thixotropy for etching of enamel and dentin.

FANTESTIC® ETCH is delivered in 2 ml-syringes and can be easily and precisely applied on the cavity surfaces with the bended application cannulas (REF ESA3071).

Indications

FANTESTIC® ETCH is according to the Total-Etch-Technique (Etch & Rinse) for enamel and dentin indicated for

- an adhesive restoration technique for composite restorations,
- pit & fissure sealings and
- adhesive luting of inlays, onlays, crowns and bridges with indicated composite luting materials.

Contraindications

At direct contact with the pulp irritations are possible. To protect the pulp areas close to the pulp should be covered with a thin layer of calciumhydroxide.

Warnings

FANTESTIC® ETCH contains phosphoric acid and is corrosive. Avoid contact of the etching gel with soft tissues, skin or eyes. Protective safety goggles for patient, dentist and assistants are recommended. In case of contact with eyes, rinse immediately with plenty of water for at least 15 minutes and consult a physician/ophthalmologist. If there is an accidental skin contact, flush immediately with soap and numerous amounts of water.

Recommended procedures

1. Preparation etching gel

Screw off the screw cap, control the opening and set up an enclosed application cannula. Store the cap to reseal the syringe after use.

Note: The application cannula might be bend in a slight arc, if desired, to facilitate the application.

Handling precaution: To get a feeling of application squeeze out the first small drop very softly on a pad to ensure proper handling and prevent "shooting" out of the etching gel by entrapped air.

2. Total-etch-technique (Etch & Rinse)

The etching gel FANTESTIC® ETCH will be applied precisely beginning with the enamel bevels onto enamel and dentin and conditioned for at least 15 s on enamel and 15 s on dentin. For teeth of 1. dentition or enamel with high fluoride content let FANTESTIC® ETCH effect correspondingly longer. Then rinse for about 20 s with water and dry the enamel with oil- and water-free air. The dentin must not dry out. The etched enamel must appear chalky white.

Precaution at/after etching: It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as described above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

After use waste the disposable application cannula and screw on the original cap to reseal the syringe.

The application cannulas are for single use only.

3. Application of an adhesive system

Then an adhesive can be applied on the etched saliva-free enamel and dentine surfaces according to the instructions of the manufacturer.

4. Filling treatment

Refer to manufacturers instructions for use for the filling treatment.

5. Storage

Do not store above 77 °F! Avoid exposure to sunlight.

Precautions to application, product life and/or storage

FANTESTIC® ETCH contains phosphoric acid and is corrosive. Avoid contact of the etching gel with soft tissues, skin or eyes. Protective safety goggles for patient, dentist and assistants are recommended. In case of contact with eyes, rinse immediately with plenty of water for at least 15 minutes and consult a physician/ophthalmologist. If there is an accidental skin contact, flush immediately with soap and numerous amounts of water.

Keep attention on the material safety data sheet (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiry date.

Keep out of the reach of children!

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.