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FANTESTIC® DESI NT

CE 0482

**Hersteller:**R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com**FANTESTIC® DESI NT**

REF FDF3301

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### FANTESTIC® DESI NT REF FDF3301

#### Description, indication and composition

FANTESTIC® DESI NT is a light-cure one-component adhesive, which acts at the same time as a desensitizer.

FANTESTIC® DESI NT is designed for a strong bonding of light-cure composites and compomers on etched dentin and enamel and on non-precious and precious metals. The strong adhesion to enamel and dentin proceeds on principles similar to that occurring with glass ionomer cements. Because of the methacrylate polycarboxylic acids a long lasting strong adhesive strength and a good biocompatibility are attained.

Due to the application as a desensitizer in cervical regions and free dentine areas during and after dental treatments all post-operative sensitivities will be eliminated. FANTESTIC® DESI NT is compatible for the tissue and does not irritate the gingiva.

The desensitizing FANTESTIC® DESI NT has good wetting properties. It is ethanol-based and hydrophilic. According to the wet-bonding technique FANTESTIC® DESI NT can be used on slightly moist dentin surfaces.

#### Recommended procedures

##### 1. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

##### 2. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction. If possible, margins should have a slight (0.5-1.0 mm) bevel placed in the enamel to increase the surface area for higher bond strength.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, Cal de Luxe®, (Ca(OH)<sub>2</sub>, "ready-to-use" in a portion blister, REF CDL2503) according to the instructions for use or with a bioactive pulp capping material, like ReViCal (REF RVC1002) according to the instructions for use.

##### 3. Total-Etch-Technique (Etch & Rinse)

Apply FANTESTIC® ETCH (REF EGS3000) precisely first on enamel and then on dentin for 15 s (reaction time enamel at least 15 s, dentin 15 s). For teeth of 1. dentition or enamel with high fluoride content let FANTESTIC® ETCH effect longer. Then rinse for about 20 s with water and dry the enamel with water- and oil-free air. The dentin should be maintained in a visibly moist condition, which is important for the function of FANTESTIC® DESI NT. The etched enamel should appear chalky white. If this is not the case, repeat the etching procedure of the enamel using a shorter reaction time.

Attention: In combination with compomers sometimes etching is not recommended, but it improves the adhesion on tooth.

Etching precaution: It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid an etching gel contact with oral soft tissues, eyes and skin. If an accidental contact occurs, flush immediately with copious amounts of water.

##### 4. Application of FANTESTIC® DESI NT in filling therapy

Apply FANTESTIC® DESI NT generously with a MICROBRUSH (REF MPB3981) onto the enamel and dentin surfaces for 30 s with agitation. The material should build a homogeneous layer. Excess material must be removed carefully. Dry cautiously with water and oil-free air for about 15 s to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentin.

Note: If FANTESTIC® DESI NT will not be used immediately, place it dispensed in subdued light to prevent premature polymerization by incident light. FANTESTIC® DESI NT will not self-cure.

##### 5. Light-cure

The FANTESTIC® DESI NT-layer will be light-cured for **20 s** before a second layer will be applied, see section 6.

FANTESTIC® DESI NT can be light-cured with commercial polymerization units (halogen or LED) with a wave length of 400-500 nm and with a light intensity > 1000mW/cm<sup>2</sup>.

The ambient light of the dental lamp may start polymerization of the FANTESTIC® DESI NT.

##### 6. Second application

The described first application according to sections 4 and 5 should be repeated for reaching a maximum adhesive strength. This is important under amalgam fillings, as FANTESTIC® DESI NT does not harden by itself.

##### 7. Placing the filling

Then continue working immediately, in accordance with the manufacturer's instructions for use for filling materials.

##### 8. Application of FANTESTIC® DESI NT as a desensitizer

Apply FANTESTIC® DESI NT generously with a MICROBRUSH (REF MPB3981) onto the enamel and dentin surfaces for 30 s with agitation. The material should build a homogeneous layer. Excess material must be removed carefully. Dry cautiously with water and oil-free air for about 15 s to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentin.

The FANTESTIC® DESI NT-layer will be light-cured for **20 s** before a second layer will be applied.

##### 9. Second application of FANTESTIC® DESI NT as a desensitizer

The described first application according to section 8 should be repeated for reaching a maximum effect.

##### 10. Storage

Do not store above 77 °F! Avoid exposure to sunlight.

#### Precautions to application, product life and/or storage

Avoid contact of the etching gel and the desensitizer with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water. In the case of contact with eyes, seek additionally medical advice.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Do not use FANTESTIC® DESI NT in case of allergic reactions. Avoid contact of FANTESTIC® DESI NT with soft tissues, skin or eyes. The contact with eyes may cause severe eye damage. Wear eye protection. In case of contact with eyes, rinse immediately with copious amounts of water and consult a physician. Commercial medical gloves do not protect against a sensitizing effect of methacrylates.

Expiry date see label. Do not use after expiry date.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

#### Warranty

R-dental Dentalerzeugnisse GmbH warrants that this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH assumes no other liability and does not grant any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for the application and the appropriate use of the product according to its intended purpose. If any defects occur to the product within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

#### Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal situation, including warranty, contract, negligence or intent.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.