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FANTESTIC® FLOWFILL SE

CE 0482

Hersteller:R-dental Dentalmaterialien GmbH · Winterhuder Weg 88 · 22085 Hamburg
T 040-30707073-0 · F 040-30707073-73 · I www.r-dental.com**FANTESTIC® FLOWFILL SE**

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FANTESTIC® FLOWFILL SE

REF FFS31XX (in syringes, XX depends on shade)

Description

FANTESTIC® FLOWFILL SE is a flowable, self-adhesive and light-cure composite for anterior and limited posterior restorations and for sealings.

Characteristics and composition

Beside its good mechanical properties, the physical properties of FANTESTIC® FLOWFILL SE are comparable to conventional flowable composites. The optimized flow behavior enables an easy application. The self-adhesive properties of FANTESTIC® FLOWFILL SE require no etching, priming and bonding of enamel and dentin in the restorative therapy. Only for an application as a pit and fissure sealant, enamel etching for cleaning is recommended.

FANTESTIC® FLOWFILL SE is based on methacrylates and inorganic filler particles of a size of 0.05-1.0 µm. The total filler load is 59 wt.-% and 45 vol.-%.

The radiopacity of FANTESTIC® FLOWFILL SE is 150 % aluminium (radiopacity of enamel: ~ 200 % aluminium, of dentin: ~ 100 % aluminium).

FANTESTIC® FLOWFILL SE meets the requirements of DIN EN ISO 4049, type 1, class 2, group 1.

Indications

FANTESTIC® FLOWFILL SE is indicated for

- small restorations of class I,
- lining of class I and class II restorations,
- pit & fissure sealings,
- extended pit & fissure sealings and
- filling of undercuts.

Side effects

Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of calcium hydroxide or a bioactive pulp capping material, see section 4.

Incompatibility with other materials

Do not use FANTESTIC® FLOWFILL SE with eugenol containing materials, because eugenol inhibits the polymerization of composites. Also the storage of composites in proximity of eugenol containing products is harmful. Do not let unpolymerized composite come in contact with eugenol containing products.

Recommended procedures

1. Preparation

Prior to the preparation a cleaning of the tooth with a fluoride-free paste (e. g. fluor pumice with water) is recommended.

2. Shade matching

Ascertain the tooth shade while teeth are still moist and select the appropriate FANTESTIC® FLOWFILL SE shade (FANTESTIC® FLOWFILL SE-products are available in VITA®-shades).

3. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

4. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction. If possible, margins should have a slight (0.5-1.0 mm) bevel placed in the enamel to increase the surface area for higher bond strength.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, Cal de Luxe®, (Ca(OH)₂) according to the instructions for use or with a bioactive pulp capping material, like ReViCal® (REF RVC100X) according to the instructions for use.

5. Application of the flowable composite FANTESTIC® FLOWFILL SE

First screw off the original-cap of the syringe and substitute it by an application cannula. Store the screw original-cap. Squeeze out the first small drop on a pad to ensure safe handling.

After use the application cannula will be removed and again the original-cap replaced.

The application cannulas are for single use only.

5 a. Small class I restorations and lining of class I and II restorations

FANTESTIC® FLOWFILL SE will be applied with the application cannula in a thin layer onto the prepared areas. By using a MICRO BRUSH (REF MPB3981) FANTESTIC® FLOWFILL SE will be worked onto the entire cavity walls and beveled areas under slight pressure for 20 s. Excess material will be removed around the margins with a microbrush.

FANTESTIC® FLOWFILL SE will be light-cured for **20 s** for lighter shades (e. g. A2) and **30 s** for darker shades (e. g. A3).

After lining of the cavity walls and the beveled areas the restoration will be build up with FANTESTIC® FLOWFILL SE in layers of max. 2 mm.

Light-cure each increment separately. FANTESTIC® FLOWFILL SE will be light-cured for **20 s** for lighter shades (e. g. A2) and **30 s** for darker shades (e. g. A3).

5 b. Pit and fissure sealing

Before sealing the enamel will be cleaned with a fluoride-free paste and rinsed thoroughly, dried in a water and oil-free air stream for 5 s leaving a slightly moist shining surface on the enamel.

Note: In addition to the cleaning as described above, etching of enamel with a 37 % phosphoric acid etching gel is recommended. Etch the enamel surface for 15-20 s, rinse for 10 s and dry in a water and oil-free air stream.

FANTESTIC® FLOWFILL SE will be placed with an application cannula in a thin layer directly onto the prepared areas. By using a MICRO BRUSH (REF MPB3981) FANTESTIC® FLOWFILL SE will be worked onto the prepared areas under slight pressure for 20 s. Excess material will be removed around the margins with a micro brush.

FANTESTIC® FLOWFILL SE can be light-cured for **20 s** for lighter shades (e. g. A2) and **30 s** for darker shades (e. g. A3). Hold the light emission window as close as possible to the filling material. The occlusion will be checked and corrected, if necessary.

6. Finishing and high gloss polishing

Depending on the shape of the area contour FANTESTIC® FLOWFILL SE can be contoured with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

Precautions to application, product life and/or storage

Avoid contact of the calcium hydroxide Cal de Luxe® and pulp capping material ReViCal with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

FANTESTIC® FLOWFILL SE can be light-cured with commercial polymerization units with a light intensity > 500 mW/cm² (Halogen) or > 1000 mW/cm² (LED) with a wavelength of 400-500 nm. The ambient light of the dental lamp may start polymerization of the composite.

Do not adjust the viscosity of the composite by using any resin. Close the syringe immediately after use to avoid exposure to light.

Do not use FANTESTIC® FLOWFILL SE in case of allergic reactions. Avoid contact of FANTESTIC® FLOWFILL SE with soft tissues, skin or eyes. The contact with eyes may cause severe eye damage. Wear eye protection. In case of contact with eyes, rinse immediately with copious amounts of water and consult a physician. Commercial medical gloves do not protect against a sensitizing effect of methacrylates.

Close syringe immediately after use to avoid exposure to light.

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F! Avoid exposure to sunlight.

*VITA is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

FANTESTIC, Cal de Luxe and ReViCal are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.