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FANTESTIC® SEAL

CE 0482

**Hersteller:**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

**FANTESTIC® SEAL**

REF FSS3920 (Farbe semiopak)

REF FSS3930 (Farbe weiß)

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REF FSS3920 (Colour semi-opaque)

REF FSS3930 (Colour white)

### Description, indication and composition

FANTESTIC® SEAL is a light-cure, fluoride containing one-component pit and fissure sealant in colour semi-opaque or white. The white colour allows a controlled application and facilitates the visual observation of the sealing, while the semi-opaque colour is more teeth like.

With the help of an etching gel, like FANTESTIC® ETCH (REF EGS3000), micro-fine retentions can be generated on the enamel surface, so that the adhesion of FANTESTIC® SEAL in fissures, pits and on the surface can be increased.

FANTESTIC® SEAL meets the requirements of DIN EN ISO 6874, class 2.

### Recommended procedures

#### 1. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

#### 2. Cleaning

The occlusal surfaces of the teeth to be sealed must be thoroughly cleaned.

#### 3. Total-Etch-Technique (Etch & Rinse)

The etching gel FANTESTIC® ETCH is applied according to the instructions for use on the surfaces and kept off from dentin. Leave FANTESTIC® ETCH in place for 30 s. Then flush thoroughly with water and dry with water- and oil-free air. The etched enamel should have a chalky white appearance.

Precaution: It is essential that the etched enamel is not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above described under 3. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

During the application with FANTESTIC® SEAL the areas should be kept dry with rubber dam.

#### 4. Application of the sealant FANTESTIC® SEAL

For application with an attached application cannula (REF FFK3281) first screw off the cap of the syringe and substitute it by an application cannula by turning it clockwise. Store the screwed cap.

FANTESTIC® SEAL will be applied with an attached application cannula cautiously directly onto the etched enamel and/or into the fissures and spread with a suitable instrument, brush or similar instrument.

After use the application cannula of the FANTESTIC® SEAL syringe will be removed and again the original-cap replaced. Because of hygienic reasons the application cannulas are for single use only.

For application with a brush FANTESTIC® SEAL will be applied directly onto the etched enamel and spread thinly with a brush. Excess material will be removed.

Each segment will be light-cured for 20 s with commercial polymerization units (halogen or LED). The spectra of the light source should range between 400-500 nm and a light intensity of at least 1,000 mW/cm<sup>2</sup>.

After polymerization, adjust occlusion and inspect sealant for complete coverage. In case of incomplete coverage reapply FANTESTIC® SEAL in the same manner.

The sealed areas are then polished with conventional instruments.

### Precautions to application, product life and/or storage

Avoid contact of the etching gel FANTESTIC® ETCH with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water. In case of contact with eyes, rinse immediately with copious amounts of water and consult a physician.

Pay attention to the Material Safety Data Sheets (MSDS >> [www.r-dental.com](http://www.r-dental.com)).

The ambient light of the dental lamp may start polymerization of the composite.

Do not adjust the viscosity of FANTESTIC® SEAL by using any resin.

Do not use FANTESTIC® SEAL in case of allergic reactions. Avoid contact of FANTESTIC® SEAL with soft tissues, skin or eyes.

Commercial medical gloves do not protect against methacrylates.

Expiration date see label. Do not use after expiration date.

Do not store above 77 °F! Avoid exposure to sunlight.

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

**Caution:** For dental use only.

### Warranty

R-dental Dentalerzeugnisse GmbH warrants that this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH assumes no other liability and does not grant any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for the application and the appropriate use of the product according to its intended purpose. If any defects occur to the product within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

### Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal situation, including warranty, contract, negligence or intent.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.