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R-SI-LINE® HARD-BITE

Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com**R-SI-LINE® HARD-BITE**

REF HBK1076

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REF HBK1076

Description and indication

R-SI-LINE® HARD-BITE in automixing cartridges is an extremely fast and extremely hard and precise bite registration material. It can be directly and bubblefree syringed onto the occlusal areas. R-SI-LINE® HARD-BITE can be used for centric occlusions and as a key material e. g. for taking intraoral registrations with needle points and is excellent applicable for bite-plates for use of face bow transfers.

Characteristics and composition

R-SI-LINE® HARD-BITE is an addition curing, syringeable registration material on vinylpolysiloxane base. R-SI-LINE® HARD-BITE is thixotropic, very suitable for cutting and contouring, inflexible and because of its consistency not crumbly. The registrations of R-SI-LINE® HARD-BITE are permanently dimensionally stable. The thixotropic properties guarantee a high stability on the occlusal areas, so that an undesired flow into the approximal areas is prevented and an easy removal is allowed.

Recommended procedures**1. Preparing the auto-mixing cartridge/R-SI-LINE® MIXING PISTOL N**

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE MIXING CANNULA. The mixing pistol is ready for application. For direct, broaden application onto the teeth use the R-SI-LINE BITE-TIPS (REF BRT1000).

2. Registration

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The automatically mixed R-SI-LINE® HARD-BITE is applied directly onto the occlusal areas or onto the bite-plate. The occlusion should be done within 20 s (working time). The registrate is hardened after 45 s in mouth can be taken out. The filled MIXING CANNULA functions as a seal until the next application.

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

Attention: Do not use latex gloves and do not contact the occlusal areas with the latex gloves, this could be inhibit the setting.

3. Articulator

R-SI-LINE® HARD-BITE allows an exact control during the fixation of the gypsum model in the articulator.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Do not use latex gloves and do not contact the occlusal areas with the latex gloves, this could be inhibit the setting.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Expiration date see label. Do not use after expiration date.

Do not store above 77 °F! Avoid exposure to sunlight.

Keep out of the reach of children!

The registrates can be stored over months without shrinkage.

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time:	< 20 s
Recommended time in the mouth:	≥ 45 s
Time (start mixing - withdrawal from mouth):	≥ 65 s

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.