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**R-SI-LINE® MEDIUM****Hersteller:**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

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REF MEK1030SH (R-SI-LINE MEDIUM SH (super hydro))
REF MEK1030SHFS (R-SI-LINE MEDIUM SH FS (super hydro, fast set))

Description and indication

R-SI-LINE® MEDIUM SH (- SH FS) in auto-mixing cartridges is used as a correction material for the sandwich-technique (one step) in conjunction preferred with the base impression material R-SI-LINE® PUTTYSOFT (- FS, - SH, - SH FS).

R-SI-LINE® MEDIUM SH (- SH FS) can also be used for the sandwich-technique (one step) for example with R-SI-LINE® PUTTY MATIC, R-SI-LINE® HEAVY MATIC or R-SI-LINE® HEAVY KINGSIZE.

Characteristics

R-SI-LINE® MEDIUM SH (- SH FS) is a regular viscosity addition curing precision impression material on vinylpolysiloxane base. It can be directly and bubblefree applied onto the teeth or the tray.

It is very thixotropic and shows a controlled flow behavior. Thus it stays in the position on the tray, but under pressure it flows.

R-SI-LINE® MEDIUM SH (super hydro) and R-SI-LINE® MEDIUM SH FS (super hydro, fast set) are very hydrophilic and show a finest detail reproduction and an excellent detail reproduction of subgingival areas.

R-SI-LINE® MEDIUM SH (- SH FS) meets the requirements ISO 4823, Type 2 (regular consistency - medium bodied).

Application

1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (Order-No.: MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE MIXING CANNULA. The mixing pistol is ready for application.

For direct fine application of the R-SI-LINE® MEDIUM onto the prepared areas additionally the R-SI-LINE® INTRA-ORAL-TIPS L should be used.

For fine applications instead of R-SI-LINE® MIXING CANNULA LN (Order-No.: MIK1099) + INTRA-ORAL-TIP L (Order-No.: IOT2001) a NEEDLE MIXING CANNULA LNN (Order-No.: MIK1099N) with an integrated, curved application needle is available.

2. Impression taking

Before application a small amount of material (size of a pea) should be pressed out and should be disposed immediately.

Load the auto-mixed R-SI-LINE® MEDIUM SH (- SH FS) onto the R-SI-LINE® SILICONE ADHESIVE (Order-No.: SIA1070) coated tray and/or onto the teeth or the prepared areas and partially/voluminous on top of the base material (- PUTTYSOFT, - FS, - SH, - SH FS).

Insert the tray with R-SI-LINE® MEDIUM SH into the mouth not later than 2 min 15 s (the fast set R-SI-LINE® MEDIUM SH FS not later than 1 min 30 s) after extruding.

Maintain the position for R-SI-LINE® MEDIUM SH at least 3 min (the fast set R-SI-LINE® MEDIUM SH FS at least 2 min) in the mouth.

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

3. Casting and disinfection

The impression should be stored at normal room temperature (< 25 °C, 78 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations.

The impressions can be disinfected with standard disinfection solutions (HIV ...).

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base- and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Expiry date see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® MEDIUM SH (- SH FS) with standard condensation reaction type silicone impression materials.

Do not use latex gloves. These could inhibit the hardening reaction of the A-silicone in the case of contamination.

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time (normal set):	< 2 min 15 s
Working time incl. mixing time (fast set):	< 1 min 30 s
Recommended time in the mouth (normal set):	≥ 3 min
Recommended time in the mouth (fast set):	≥ 2 min
Time (start mixing - withdrawal from mouth) (normal set):	≥ 5 min 15 s
Time (start mixing - withdrawal from mouth) (fast set):	≥ 3 min 30 s
Dimensional change after 24 hours:	< 0.1 %
Elastic recovery:	> 99.4 %
Strain in compression:	about 3.5 %
Shore A-hardness:	about 55

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.