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P.U.M.A. hard

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

P.U.M.A. hard

REF PUM5000 (P.U.M.A. hard, Systempackung)

REF PUK5010 (P.U.M.A. hard, Nachfüllpackung)

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REF PUM5000 (P.U.M.A. hard, system kit)

REF PUK5010 (P.U.M.A. hard, refill kit)

REF PUA5020 (P.U.M.A. hard ADHESIVE, refill kit)

Description and indication

P.U.M.A. hard is a hard, cold-curing relining material for partial -, total - or interim prostheses (dentures) on polymethyl methacrylate basis (PMMA).

P.U.M.A. hard shows an excellent long-term adhesion on dentures on polymethyl methacrylate basis (PMMA) or combined dentures in combination with P.U.M.A. hard ADHESIVE.

It can be used directly/chairside or indirectly in the lab. P.U.M.A. hard is free of methyl methacrylate.

P.U.M.A. hard is delivered in KwikkMix ®-System 10:1.

Recommended procedure

1. Preparing the double chamber syringe

Before first use screw off the original cap and waste it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally. Then the mixing cannula FANTESTIC ® MISCHKANÜLE MPN 2 (REF MIK1046) is put on and locked by turning it 90 ° clockwise. The double chamber syringe is ready for application.

2. Preparation of the denture areas to be relined

After a functional test (occlusion control) the areas to be relined (denture borders and/or denture base) will be roughened with a suitable bur, cleaned and dried.

Attention: Areas which should not be relined must be insulated, e. g. with vaseline. Anchors, clasps and attachments of partial dentures have to be blocked out with an easy flowing silicone or wax with a low melting point.

Then P.U.M.A. hard ADHESIVE (REF PUA5020) will be brushed extraorally as a thin coat on the denture areas and dried in the air for about 15 s.

Attention: P.U.M.A. hard ADHESIVE should only be used extraorally.

Do not contaminate the sticky oxygen inhibition layer, because this layer will be in adhesive contact to the relining material.

Attention: Store the P.U.M.A. hard ADHESIVE-bottle vertically. Methacrylate is volatile. Immediately close the bottle after use.

3. Application of the relining material P.U.M.A. hard (chairside)

Before every application a small amount of material (size like rice corn to pea) should be pressed out and should be disposed of immediately.

Then apply the automatically mixed P.U.M.A. hard uniformly on the denture borders and/or denture base conditioned with P.U.M.A. hard ADHESIVE and insert the denture within 1 min into the mouth of the patient (working time at 23 °C (74 °F)).

Attention: Avoid that excess material will float behind the A-line (region from hard to soft palate) into the throat.

Then let the patient close his occlusion (icp) for about 30 s and carry out active and passive functional excursions for about 1 min 30 s.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

3 a. Partial or complete dentures with undercuts

Intraoral: Remove the denture from the mouth after about 2-2,5 min of starting the application. Remove the excess relining material with scalpel or scissors. Then introduce the denture for final occlusion control and let it cure for about 2-3 min.

Extraoral: The complete curing can be accelerated in warm water or in a polymerizing device at 40 °C (104 °F) (to inhibit the oxygen inhibition layer, do not let it cure in the air).

3 b. Partial or complete dentures without undercuts

The excess material will be removed before final polymerization.

Intraoral: Remove the denture from the mouth after about 2-2,5 min of starting the application. Remove the excess relining material with scalpel or scissors. Then introduce the denture for final occlusion control and let it cure for about 2-3 min.

Extraoral: Optional the complete curing can be accelerated in warm water or in a polymerizing device at 40 °C (104 °F) (to inhibit the oxygen inhibition layer, do not let it cure in the air).

The sticky oxygen inhibition layer can easily be removed with alcohol.

4. Application of the relining material P.U.M.A. hard (labside)

Embed the denture to be relined into a flask with counter. Then remove the denture from the flask and prepare as described under section 1 and section 2. Then isolate the model with an usual alginate based insulation and apply the required automatically mixed P.U.M.A. hard uniformly (with an excess layer thickness of 1 mm) on the denture borders and/or denture base to be relined.

Close flask with counter and press with light pressure for approximately 30 minutes (at 23 °C or 74 °F) or place flask in a pressure pot at 40-45 °C (104-113 °F) for approximately 10 minutes.

Then deflask the relined denture and trim and polish as described under point 5.

5. Finishing the relined denture

The oxygen inhibition layer can be removed with alcohol. The relined areas can be finished with suitable instruments (burs, polishers and so on).

Attention: Do not breathe polishing dust. Use suitable mouth protective device or aspiration!

6. Precautions

P.U.M.A. hard is free of methyl methacrylate, but contains other methacrylates. With susceptible patients, sensitization to P.U.M.A. hard cannot be excluded. P.U.M.A. hard should not be used any more, if allergic reactions are observed. Do not use for patients with allergic reactions against acrylates. If swallowed immediately consult a physician.

Further on avoid contact of P.U.M.A. hard ADHESIVE and P.U.M.A. hard with skin or eyes. If there is an accidental contact with skin, flush immediately with soap and numerous amounts of water. If there is an accidental contact with eyes, flush immediately with numerous amounts of water and consult a physician if necessary.

7. Storage

Do not store above 25 °C (77 °F). Avoid exposure to sunlight.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Do not re-use the original cartridge cap.

Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size like rice corn to pea) should be pressed out and should be disposed of immediately.

The filled MIXING CANNULA functions as a seal until the next application.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Shelf life see label. Do not use after expiry date.

The double chamber syringe should be stored horizontally (laying).

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.