R-SI-LINE® SILIKON-ADHÄSIV

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Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

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REF SIA1070 (R-SI-LINE® SILICONE-ADHESIVE)**Description and indication/intended use**

R-SI-LINE® SILICONE-ADHESIVE is an universal adhesive for adhering A- and C-silicone impression materials and impression trays made of metal or individualized methacrylate impression trays.

R-SI-LINE® SILICONE-ADHESIVE inhibits an uncontrolled separation of the impression material from the bottom of the tray and deformation during removal without distortion.

Note: R-SI-LINE® SILICONE-ADHESIVE can be used restricted with reduced adhesion on plastic impression trays. The adhesive cannot be used for polyether impression materials.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

The use of the product is contraindicated in the case of known hypersensitivity to the components of the product.

Patient target group

Persons who are treated during a dental procedure.

Intended users

Only for professional trained dental/dental technician users.

Recommended procedures**1. Preparation of the impression tray**

First, the impression tray will be cleaned and dried. Then before the impression taking R-SI-LINE® SILICONE-ADHESIVE will be applied in a thin layer onto the inner areas of the impression tray.

After drying at least 3 min the impression tray can be loaded with an impression material. Then the impression can be taken.

2. Cleaning of the impression tray

On a metal tray the silicone adhesive can be removed easily using acetone.

Precautions to application, product life and/or storage

Avoid contact of R-SI-LINE® SILICONE-ADHESIVE with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Apply R-SI-LINE® SILICONE-ADHESIVE only in well ventilated rooms. Do not inhale!

Pay attention to the Material Safety Data Sheet (MSDS), see www.r-dental.com.

Expiry date see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Side effects are not known to this point. The residual risk of possible hypersensitivity to individual components of the material cannot be excluded.

Disposal

Disposal must be carried out in accordance with local official regulations.

Reporting obligation

Serious incidents according to the MDR that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.