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FANTESTIC® FLOWSIVE SE

CE 0482

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • E info@r-dental.com • I www.r-dental.com

**FANTESTIC® FLOWSIVE SE**

REF FFF5155

**Produktbeschreibung**

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### FANTESTIC® FLOWSIVE SE REF FFF5155

#### Product description

FANTESTIC® FLOWSIVE SE is a simple to use, self-etching and dual-cure adhesive system for a strong adhesion of light-cure composites, compomers and resin-modified glass ionomer cements on dentin and enamel. It provides an optimal and secure adhesion to slightly moist dentin and enamel. FANTESTIC® FLOWSIVE SE bonds to light-cure, self-curing and dual-cure composites.

FANTESTIC® FLOWSIVE SE consists of two components A and B which have to be mixed in equal volumes before application.

#### Indications

FANTESTIC® FLOWSIVE SE is indicated

- for core build-ups and luting of root canal posts with self- and dual-cure composites, e. g. FANTESTIC® CORE DC or FANTESTIC® Z CORE DC,
- for luting of root canal posts with self- and dual-cure composites, e. g. FANTESTIC® FLOWFILLCEM DC,
- for luting of inlays, onlays, crowns and bridges with self- and dual-cure luting composites, e. g. FANTESTIC® FLOWFILLCEM DC and
- for composite restorations with self-, light-cure and dual-cure composites.

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

In rare cases, FANTESTIC® FLOWSIVE SE can lead to sensitisation in patients with hypersensitivity to one of the components. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of pulp capping material, see section 3.

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Incompatibilities with other materials

Do not use FANTESTIC® FLOWSIVE SE in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store FANTESTIC® FLOWSIVE SE in proximity of eugenol containing materials. Unpolymerized FANTESTIC® FLOWSIVE SE should not come in contact with eugenol containing materials.

#### Application

##### 1. Isolation

A dry area and use of rubber dam is recommended.

##### 2. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction.

##### 3. Pulp protection

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, like Cal de Luxe®, according to the instructions for use or with a bioactive pulp capping material, like ReViCal® (REF RVC100X) or ReViCal® Liner (REF RVC10XX), according to the instructions for use.

##### 4. Application of the adhesive FANTESTIC® FLOWSIVE SE

One drop each of FANTESTIC® FLOWSIVE SE (REF FFF5155) Liquid A + Liquid B will be mixed light protected 1:1 within the enclosed FANTESTIC® Mixing moulds (REF FFF5180) for about 5-10 s.

Attention: Do not use component Liquid A or Liquid B alone and do not interchange the lids of the bottles of Liquid A and B, because a cross contamination of the components can occur.

##### 4.1. Application of the adhesive for core-build-ups, for adhesive luting of inlays, onlays, crowns and bridges and for composite fillings

FANTESTIC® FLOWSIVE SE will be applied with a MICROBRUSH (REF MPB3981) onto the prepared and physiological slightly wet dentin - and enamel surfaces and agitate it intensively for 30 s.

It is important to ensure, that the adhesive mixture should build a homogeneous layer over the entire surface. Then aerate the solvent in FANTESTIC® FLOWSIVE SE for about 10 s in a weak oil- and waterfree air stream to distribute the adhesive and light-cure the adhesive for 20 s with commercial polymerization units with a light intensity > 500 mW/cm<sup>2</sup> (Halogen) or > 1000 mW/cm<sup>2</sup> (LED) with a wavelength of 400-500 nm.

Attention: To achieve a maximum adhesion, light-cure of the adhesive is strongly recommended.

If light-cure is absolutely impossible, FANTESTIC® FLOWSIVE SE will also cure in the self-curing mode. After application and agitation the solvent in FANTESTIC® FLOWSIVE SE must be aerated in a weak oil- and waterfree air stream.

There remain a smear layer (oxygen inhibition layer) to which the light-cure, self-curing or dual-cure composite used can be applied directly.

##### 4.2. Application of the adhesive for adhesive luting of root canal posts

The lege artis prepared root canals will be cleaned (e. g. with sodium hypochlorite solution) and rinsed. Remove excess liquid using paper points.

Then the homogeneous mixture will be applied in an abundant quantity with a brush into the slightly wet root canal and will be agitated intensively for about 15 s. Repeat this step 1-2 times. It is important to ensure that the mixture is distributed homogeneously over the entire surface. Remove excess adhesive using paper points.

Then aerate the solvent carefully for about 15 s in a weak oil- and waterfree air stream to remove all volatile components and distribute the adhesive evenly.

The mixture should be light-cured (if locally possible) for about 20 s.

##### 5. Application of a composite

According to the instructions for use a light-cure, self-curing or dual-cure composite can be applied directly.

##### 6. Storage

Do not store above 25 °C (77 °F)! Avoid exposure to sunlight. Do not use after expiry date.

##### Additional Notes/Warnings

Unpolymerized adhesive may have an irritant effect and can lead to sensitization against methacrylates.

Avoid contact of the adhesive with soft tissues, skin or eyes.

In case of contact with skin flush immediately with water and soap. In case of contact with eyes, rinse immediately with copious amounts of water and if necessary consult a physician.

Commercial medical gloves do not protect against a sensitizing effect of methacrylates.

Keep out of the reach of children!

##### Composition

Hydrophilic methacrylates, MDP, photo initiators, catalysts.

##### Disposal

Disposal of the product according to local authority regulations.

##### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

##### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

FANTESTIC, Cal de Luxe and ReViCal are registered trademarks of R-dental Dentalerzeugnisse GmbH.

##### Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

##### Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2024 R-dental Dentalmaterialien GmbH. All rights reserved.