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FANTESTIC® FLOWSIVE SE

CE 0482

Hersteller:
R-dental Dentalmaterialien GmbH · Winterhuder Weg 88 · 22085 Hamburg
T 040-30707073-0 · F 040-30707073-73 · I www.r-dental.com

FANTESTIC® FLOWSIVE SE

REF FFF5155

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REF FFF5155

Description

FANTESTIC® FLOWSIVE SE is a simple to use, self-etching and dual-cure adhesive system for a strong adhesion of light-cure composites, compomers and resin-modified glass ionomer cements on dentin and enamel. It provides an optimal and secure adhesion to slightly moist dentin and enamel. FANTESTIC® FLOWSIVE SE bonds to light-cure, self-curing and dual-cure composites.

FANTESTIC® FLOWSIVE SE consists of two components A and B which have to be mixed before application.

Indications

FANTESTIC® UniversalBond is indicated

- for luting of root canal posts with self- and dual-cure composites, e. g. FANTESTIC® FLOWFILLCEM DC,
- for luting of root canal posts followed by a core build-up with self- and dual-cure core build-up composites, e. g. FANTESTIC® CORE DC, FANTESTIC® Z CORE DC,
- for luting of inlays, onlays, crowns and bridges with self- and dual-cure luting composites, e. g. FANTESTIC® FLOWFILLCEM DC and
- for composite restorations with self-, light-cure and dual-cure composites.

Side effects

In rare cases FANTESTIC® FLOWSIVE SE may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, FANTESTIC® FLOWSIVE SE should not be used. Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of calcium hydroxide or a bioactive pulp capping material, see section 2.

Incompatibilities

Do not use FANTESTIC® FLOWSIVE SE in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store FANTESTIC® FLOWSIVE SE in proximity of eugenol containing materials. Unpolymerized FANTESTIC® FLOWSIVE SE should not come in contact with eugenol containing materials.

Recommended procedures

1. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

2 Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction. If possible, margins should have a slight (0.5-1.0 mm) bevel placed in the enamel to increase the surface area for higher bond strength.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, Cal de Luxe®, (Ca(OH)₂, "ready-to-use" in a portion blister, REF CDL2503) according to the instructions for use or with a bioactive pulp capping material, like ReViCal (REF RVC1002) according to the instructions for use.

3. Preparation of the adhesive FANTESTIC® FLOWSIVE SE

One drop each of FANTESTIC® FLOWSIVE SE (REF FFF5155) Liquid A + Liquid B will be mixed light protected 1:1 within the enclosed FANTESTIC® Mixing moulds (REF FFF5180) for about 5-10 s.

Attention: Do not use component Liquid A or Liquid B alone and do not interchange the lids of the bottles of Liquid A and B, because a cross contamination of the components can occur.

4. Application of the adhesive for core-build-ups, for adhesive luting of inlays, onlays, crowns and bridges and for composite fillings

FANTESTIC® FLOWSIVE SE will be applied with a MICROBRUSH (REF MPB3981) onto the prepared and physiological slightly wet dentin - and enamel surfaces and agitate it intensively for 30 s.

The adhesive mixture should build a homogeneous layer over the entire surface. Then aerate the solvent in FANTESTIC® FLOWSIVE SE for about 10 s in a weak oil- and waterfree air stream and light-cure the adhesive for 20 s.

FANTESTIC® UniversalBond can be light-cured with commercial polymerization units with a light intensity > 500 mW/cm² (Halogen) or > 1000 mW/cm² (LED) with a wavelength of 400-500 nm. The ambient light of the dental lamp may start polymerization of the adhesive.

Attention: To achieve a maximum adhesion, light-cure of the adhesive is strongly recommended.

If light-cure is absolutely impossible, FANTESTIC® FLOWSIVE SE will also cure in the self-curing mode. It is important to ensure that the mixture is distributed homogeneously over the entire surface. Then aerate the solvent in FANTESTIC® FLOWSIVE SE for about 10 s in a weak oil- and waterfree air stream. There remain a smear layer (oxygen inhibition layer) to which the light-cure, self-curing or dual-cure composite used can be applied directly.

Then it can be proceed immediately, see section 6.

4. Application of the adhesive for adhesive luting ("cementing") of root canal posts

Before starting the preparation, the tooth must be cleaned of residues etc. The lege artis prepared root canals will be cleaned (e. g. with sodium hypochlorite solution) and rinsed. Remove excess liquid using paper points.

Then the mixture prepared according to section 3 will be applied with a brush into the slightly wet root canal and will be agitated intensively for about 15 s. Repeat this step 1-2 times. It is important to ensure that the mixture is distributed homogeneously over the entire surface. Remove excess adhesive using paper points. Then aerate the solvent for about 15 s in a weak oil- and waterfree air stream.

The mixture should be light-cured (if locally possible) for about 20 s to get an optimal chemical adhesion.

6. Application of a composite

According to the instructions for use a light-cure (if indicated), self-curing or dual-cure composite can be applied directly.

7. Storage

Do not store above 77 °F! Avoid exposure to sunlight.

Precautions to application, product life and/or storage

Avoid contact of calcium hydroxide Cal de Luxe® and the pulp capping material ReViCal with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Do not use FANTESTIC® FLOWSIVE SE in case of allergic reactions. Avoid contact of FANTESTIC® FLOWSIVE SE with soft tissues, skin or eyes. In case of contact with skin flush immediately with water and soap. In case of contact with eyes, rinse immediately with copious amounts of water and if necessary consult a physician. Commercial medical gloves do not protect against a sensitizing effect of methacrylates.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Close the bottle immediately after use.

Expiry date see label. Do not use after expiry date.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalmaterialien GmbH warrants that this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH assumes no other liability and does not grant any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for the application and the appropriate use of the product according to its intended purpose. If any defects occur to the product within the warranty period, your exclusive remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal situation, including warranty, contract, negligence or intent.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.