FANTESTIC® APPLIKATOR SYNCRO

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Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

FANTESTIC® APPLIKATOR SYNCRO

REF FFK3405

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FANTESTIC® APPLIKATOR SYNCRO

REF FFK3405

The FANTESTIC® APPLIKATOR SYNCRO is an applicator for commercial composite tips.

Important precautions to reuse

The FANTESTIC® APPLIKATOR SYNCRO must be sterilized before the first use at the patient and after every use at the patient with steam at 273 °F (for 3 min, air steam disinfectant, keep attention on instructions for use of manufacturer).

The FANTESTIC® APPLIKATOR SYNCRO can further on be disinfected with alcohol and commercial disinfectants, which contain 1,5 % glutaraldehyd.

The treatment may last not longer than 60 min.

The FANTESTIC® APPLIKATOR SYNCRO can not be disinfected with other disinfection methods; in these cases there is no warranty.

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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FANTESTIC ist ein eingetragenes Warenzeichen der R-dental Dentalerzeugnisse GmbH.
© 2021 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE