FANTESTIC® OxyBlok

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Hersteller:R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com**FANTESTIC® OxyBlok**
REF FOS3970**Beschreibung und Indikation**

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FANTESTIC® OxyBlok is a protecting-gel. The protecting-gel inhibits the contact of oxygen on a composite layer. As a result the so called oxygen inhibition layer will be reduced or inhibited.

Because of the application of FANTESTIC® OxyBlok composite surfaces of the last layer of light-cure, self-curing or dual-cure composites can be fully polymerized at light-cure oder dual-cure polymerization.

Recommended procedures**1 a. Application of FANTESTIC® OxyBlok - filling treatment**

After an adhesive filling treatment according to the instructions for use the protecting-gel is applied in a thin layer of about 1-2 mm onto the last composite layer. Then the composite layer will be light-cured or self-cured. After this the gel will be rinsed intensively with water.

The composite surface can then be polished to high-gloss smearless finish.

1 b. Application of FANTESTIC® OxyBlok - luting treatment

After a luting treatment of restorations according to the instructions for use the protecting-gel is applied in a thin layer of about 1-2 mm onto the cementing seam. Then the composite will be light-cured or self-cured. After this the gel will be rinsed intensively with water.

The composite seam can then be worked out smearless.

Precautions to application, product life and/or storage

The application cannulas are for single use only.

Expiration date see label. Do not use after expiration date.

Do not store above 77 °F! Avoid exposure to sunlight.

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2022 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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© 2022 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
Manufacturer:R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE