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FANTESTIC® OxyBlok

MD CE 0482

**Hersteller:**

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • E info@r-dental.com • I www.r-dental.com

**FANTESTIC® OxyBlok**  
REF FOS3970

**Produktbeschreibung**

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**FANTESTIC® OxyBlok**  
REF FOS3970

**Product description**

FANTESTIC® OxyBlok is a protective-gel and serves as an oxygen barrier during the curing of light-curable and self-curing composites. This effectively reduces the so-called oxygen inhibition layer.

**Indications/Intended use**

FANTESTIC® OxyBlok is indicated as a protecting-gel.

**Performance features**

The performance features of the product meet the requirements of the intended use.

**Contraindications**

Not known.

**Patient target group**

Persons who are treated during a dental procedure.

**Intended users**

This medical device should only be used by a professionally trained dental practitioner.

**Application****1 a. Application of FANTESTIC® OxyBlok - filling treatment**

After an adhesive filling treatment according to the instructions for use the protecting-gel is applied in a thin layer of about 1-2 mm onto the last composite layer. Then the composite layer will be light-cured or self-cured.

After this the gel will be rinsed intensively with water.

The composite surface can then be polished to high-gloss smearless finish.

**1 b. Application of FANTESTIC® OxyBlok - luting treatment**

After a luting treatment of restorations according to the instructions for use the protecting-gel is applied in a thin layer of about 1-2 mm onto the cementing seam. Then the composite will be light-cured or self-cured.

After this the gel will be rinsed intensively with water.

The composite seam can then be worked out smearless.

**4. Storage**

Do not store above 25 °C (77 °F)! Avoid exposure to sunlight. Do not use after expiry date.

**Additional Notes/Warnings**

Keep out of the reach of children!

**Disposal**

Disposal of the product according to local authority regulations.

**Reporting obligation**

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

**Warranty**

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

**Limitation of Liability**

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2024 R-dental Dentalmaterialien GmbH. All rights reserved.