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R-SI-LINE® HEAVY EXACT K

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

REF HEK1026 (R-SI-LINE® HEAVY EXACT K)

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REF HEK1026 (R-SI-LINE® HEAVY EXACT K)

Description and indication

R-SI-LINE® HEAVY EXACT K in auto-mixing cartridges is a high viscosity precision impression material and is used preferable as a base impression material for one step, two phase impressions in combination with R-SI-LINE® MEDIUM SH (REF MEK1030SH) und R-SI-LINE® MEDIUM SH FS (REF MEK1030SHFS) as correction impression materials.

Characteristics

R-SI-LINE® HEAVY EXACT K is an addition curing precision impression material on vinylpolysiloxane base (A-silicone). It can be directly and bubblefree syringed onto the teeth or the impression tray.

R-SI-LINE® HEAVY EXACT K is hydrophilic and thus improves the wettability and shows an excellent detail reproduction especially of subgingival areas.

R-SI-LINE® HEAVY EXACT K meets the requirements of DIN EN ISO 4823 Type 1 (heavy consistency - heavy bodied).

Application

1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE MIXING CANNULA. The mixing pistol is ready for application.

2. Impression taking

Before every application a small amount of material (size of a pea) should be pressed out and should be wasted immediately.

Load the auto-mixed R-SI-LINE® HEAVY EXACT K onto the R-SI-LINE® SILICONE ADHESIVE (REF SIA1070) coated tray.

At the same time the automatically mixed R-SI-LINE®-correction material can be applied directly onto the teeth or the prepared areas and partially/bulky onto the base material R-SI-LINE® HEAVY EXACT K.

Insert the tray into the mouth not later than 2 min after extruding (working time at 74 °F). Maintain the position of the impression for at least 3 min in the mouth.

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

3. Casting and disinfection

The impression should be stored at normal room temperature (< 25 °C, 77 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations.

The impressions can be disinfected with standard disinfection solutions.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base- and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Keep attention on the material safety data sheet (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® HEAVY EXACT K with standard condensation reaction type silicone impression materials.

Do not use Latex gloves. These could inhibit the hardening reaction of the A-silicone in the case of contamination.

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time:	< 2 min
Recommended time in the mouth:	≥ 3 min
Time (start mixing - withdrawal from mouth) (- MONO S):	≥ 5 min
Dimensional change after 24 hours:	< 0.1 %
Elastic recovery:	> 98.6 %
Strain in compression:	about 2 %
Shore-A-Hardness:	about 72

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.