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R-SI-LINE® HEAVY KINGSIZE

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

REF HKK1051 (R-SI-LINE® HEAVY KINGSIZE)

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#### REF HKK1051 (R-SI-LINE® HEAVY KINGSIZE)

#### Description and indication

R-SI-LINE® HEAVY KINGSIZE in 75 ml-automix cartridges is an automatically mixed medium viscosity addition curing impression material on vinyl polysiloxane base (A-Silicone).

R-SI-LINE® HEAVY KINGSIZE is used as a very hydrophilic base impression material for the sandwich-technique (one step) in conjunction with R-SI-LINE®-correction impression materials (e. g. with R-SI-LINE® MEDIUM SH (super hydro, REF MEK1030SH) or R-SI-LINE® LIGHT SH (super hydro, REF LIK1040SH)).

#### Characteristics

R-SI-LINE® HEAVY KINGSIZE is thixotropic and super hydrophilic and therefore compatible with moist surfaces and preferably used for impressions in conjunction with the super hydrophilic R-SI-LINE®-correction impression materials (super hydro).

R-SI-LINE® HEAVY KINGSIZE meets the requirements of ISO 4823 Type 2 (medium consistency - medium bodied).

#### Application

##### 1. Preparation of the tray

It may be used perforated or non-perforated metal or plastic trays. In order to improve the adhesion of R-SI-LINE® HEAVY KINGSIZE to the tray, the R-SI-LINE® SILICONE-ADHESIVE (REF SIA1070) should be applied as a thin even film.

##### 2. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE® MIXING CANNULA. The mixing pistol is ready for application.

##### 3. Impression taking

Before every application a small amount of material (size of a pea) should be pressed out and should be wasted immediately.

The automatically mixed impression material is loaded directly into the prepared tray. Insert the loaded tray into the mouth within 2 min after start of mixing together with the R-SI-LINE®-correction material (e. g. R-SI-LINE® MEDIUM SH, REF MEK1030SH) (working time at 74 °F). Apply the correction material directly onto the top of the base material in the tray and on the prepared areas into the mouth. Maintain these two-phase impressions with normal set components at least 3 min in the mouth (hardening time).

Time mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

##### 4. Casting and disinfection

The impression should be stored at normal room temperature (< 25 °C, 77 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations.

The impressions can be disinfected with standard disinfection solutions.

#### Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Keep attention on the material safety data sheet (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® HEAVY KINGSIZE with standard condensation reaction type silicone impression materials.

Do not use Latex gloves. These could inhibit the hardening reaction of the A-silicone in the case of contamination.

#### Technical data

|  |             |
|--|-------------|
| Mixing time:                                 | automix     |
| Working time incl. mixing time:              | < 2 min     |
| Recommended time in mouth:                   | ≥ 3 min     |
| Time (start mixing - withdrawal from mouth): | ≥ 5 min     |
| Dimensional change after 24 hours:           | < 0.1 %     |
| Elastic recovery:                            | > 99.8 %    |
| Strain in compression:                       | about 2.5 % |
| Shore A-hardness:                            | about 65    |

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

**Caution:** For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

#### Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

#### Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.