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KwikkDam

**Hersteller**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

KwikkDam (5 ml-KwikkMix ®)
Bestell-Nr.: KDS7050

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KwikkDam (5 ml-KwikkMix ®)
Order-No.: KDS7050

Description and indication

KwikkDam in the commercial 1:1-double-chamber-syringe (KwikkMix ®) is a very fast setting, addition curing silicone on a vinylpolysiloxane basis (A-silicone) with regular viscosity.

KwikkDam is used as a liquid dam and can be applied directly, bubble free and precise onto the gingival areas to protect the gingiva and approximal areas at bleaching procedures or at other indications.

KwikkDam is very thixotropic and very standable, that means that it stays on the gingival areas and can be dispensed easily and homogeneously. KwikkDam is super hydrophilic and therefore improves the wettability and adhesion to the gingival areas.

KwikkDam can be removed after the dental treatment very easily by peeling.

KwikkDam meets the requirements ISO 4823, Cat. A, Type 2 (regular viscosity).

Recommended procedures**1. Preparing the double-chamber-syringe**

First remove the original cap (do not re-use!) of the double-chamber-syringe and as a check press out a small amount of material without mixing cannula. Then screw on the MIXING CANNULA MCN (Order-No.: MIK1044). The double-chamber-syringe is ready for use.

2. Application of KwikkDam

The automatically mixed KwikkDam will be directly syringed onto the gingival areas and dispensed homogeneously.

To protect the gingiva against a bleaching gel, the gingival areas should be completely covered with KwikkDam and, if desired or necessary, the proximal interdental spaces should be closed.

3. Removal

The liquid rubber dam can be easily removed by peeling.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Do not re-use the original cartridge cap. Closing the double chamber syringe by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting.

When screwing on a new mixing cannula the double chamber syringe should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application.

Expiration date see label. Do not use after expiration date.

Do not store above 78 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

The double chamber syringe should be stored horizontally (laying).

R-SI-LINE and KwikkMix are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for defections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2016 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE