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R-SI-LINE® LIGHT

**Hersteller:**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

REF LIK1040 (R-SI-LINE® LIGHT)**REF LIK1040FS (R-SI-LINE® LIGHT FS (fast set))****REF LIK1040SH (R-SI-LINE® LIGHT SH (super hydro))****REF LIK1040SHFS (R-SI-LINE® LIGHT SH FS (super hydro, fast set))****Beschreibung und Indikation**

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REF LIK1040 (R-SI-LINE® LIGHT)

REF LIK1040FS (R-SI-LINE® LIGHT FS (fast set))

REF LIK1040SH (R-SI-LINE® LIGHT SH (super hydro))

REF LIK1040SHFS (R-SI-LINE® LIGHT SH FS (super hydro, fast set))

Description and indication

R-SI-LINE® LIGHT (- FS, - SH, - SH FS) in auto-mixing cartridges is used as a corection material for the putty-wash-technique (two steps) in conjunction with the base impression material R-SI-LINE® PUTTY (REF PUT1010) or R-SI-LINE® PUTTY FS (REF PUT1010FS) or R-SI-LINE® PUTTY MATIC (REF PMK1013).

R-SI-LINE® LIGHT (- FS, - SH, - SH FS) can also be used for the sandwich-technique (one step) for example with R-SI-LINE® PUTTYSOFT, R-SI-LINE® PUTTY MATIC, R-SI-LINE® HEAVY MATIC or R-SI-LINE® HEAVY KINGSIZE.

Characteristics

R-SI-LINE® LIGHT (- FS, - SH, - SH FS) is a low viscosity addition curing precision impression material on vinylpolysiloxane base. It can be applied directly and bubblefree onto the teeth or the tray.

It is very thixotropic and shows a controlled flow behavior. Thus it stays in the position on the tray, but under pressure it flows.

R-SI-LINE® LIGHT (- FS) is highly hydrophilic and shows a finest detail reproduction.

R-SI-LINE® LIGHT SH (super hydro) and R-SI-LINE® LIGHT SH FS (super hydro, fast set) are very hydrophilic and show an excellent detail reproduction also of subgingival areas.

R-SI-LINE® LIGHT (- FS, - SH, - SH FS) meets the requirements of ISO 4823 Type 3 (low consistency - light bodied).

Recommended procedures

1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE MIXING CANNULA. The mixing pistol is ready for application.

For direct fine application of the R-SI-LINE® LIGHT (- FS, - SH, - SH FS) onto the prepared areas additionally the R-SI-LINE® INTRA-ORAL-TIPS L (REF IOT2001) should be used.

For fine applications instead of R-SI-LINE® MIXING CANNULA LN (REF MIK1099) + INTRA-ORAL-TIP L (REF IOT2001) a NEEDLE MIXING CANNULA LNN (REF MIK1099N) with an integrated, curved application needle is available.

2. Impression taking

Before application a small amount of material (size of a pea) should be pressed out and should be wasted immediately.

Load the auto-mixed R-SI-LINE® LIGHT (- FS, - SH, - SH FS) onto the R-SI-LINE® SILICONE ADHESIVE (REF SIA1070) coated tray and/or onto the teeth or the prepared areas and partially/voluminous on top of the base material (R-SI-LINE® PUTTY, - PUTTY FS or - PUTTY MATIC/ - HEAVY MATIC or - HEAVY KINGSIZE).

Insert the tray with R-SI-LINE® LIGHT (- SH) into the mouth not later than 2 min (the fast set components - LIGHT FS, - SH FS not later than 1 min 30 s) after extruding (working time).

Maintain the position for R-SI-LINE® LIGHT (- SH) at least 3 min (the fast set components - LIGHT FS, - LIGHT SH FS at least 2 min) in the mouth (hardening time).

Times mentioned for 23 °C (74 °F). Higher temperatures shorten, lower temperatures prolong these times.

3. Casting and disinfection

The impression should be stored at normal room temperature (< 25 °C, 77 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations.

The impressions can be disinfected with standard disinfection solutions.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Expiry date see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)!

Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® LIGHT (- FS, - SH, - SH FS) with standard condensation reaction type silicone impression materials.

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time (normal set):	< 2 min
Working time incl. mixing time (fast set):	< 1 min 30 s
Recommended time in the mouth (normal set):	≥ 3 min
Recommended time in the mouth (fast set):	≥ 2 min
Time (start mixing - withdrawal from mouth) (normal set):	≥ 5 min
Time (start mixing - withdrawal from mouth) (fast set):	≥ 3 min 30 s
Dimensional change after 24 hours:	< 0.1 %
Elastic recovery:	> 99.6 %
Strain in compression:	about 4.9 %
Shore A-hardness:	about 45

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2020 R-dental Dentalerzeugnisse GmbH. All rights reserved.

Bite to Perfection

R-SI-LINE® LIGHT

**Fabricant:**R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE
T +49 40-30707073-0 • F +49 40-30707073-73 • I www.r-dental.com

REF LIK1040 (R-SI-LINE® LIGHT)

REF LIK1040FS (R-SI-LINE® LIGHT FS (fast set))

REF LIK1040SH (R-SI-LINE® LIGHT SH (super hydro))

REF LIK1040SHFS (R-SI-LINE® LIGHT SH FS (super hydro, fast set))

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R-SI-LINE est une marque déposée de R-dental Dentalerzeugnisse GmbH.
© 2020 R-dental Dentalerzeugnisse GmbH. Tous droits réservés.