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R-SI-LINE® METAL-BITE® Blue

**Hersteller:**

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

R-SI-LINE® METAL-BITE® Blue

REF MBK0475

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REF MBK0475

Description and characteristics

R-SI-LINE[®] METAL-BITE[®] Blue in automixing cartridges is an extremely fast, extremely hard and precise addition curing registration material on vinyl polysiloxane base. It can be directly and bubblefree syringed onto the occlusal areas.

R-SI-LINE[®] METAL-BITE[®] Blue can also be applied as key material e. g. for taking intraoral registrations with needle points and as an material for fixation of bite forks for use with face bows.

R-SI-LINE[®] METAL-BITE[®] Blue is scannable and therefore usable for powderfree optical registration of data (antagonist registration).

Thanks to its thixotropy, undesired flow of R-SI-LINE[®] METAL-BITE[®] Blue into dental interstices is prevented and allows an easy removal.

R-SI-LINE[®] METAL-BITE[®] Blue is very suitable for cutting and contouring, inflexible and not crumbly. Registrations of R-SI-LINE[®] METAL-BITE[®] Blue are for months dimensionally stable.

Indications/Intended use

R-SI-LINE[®] METAL-BITE[®] Blue is indicated as

- a bite registration material,
- a key material for taking intraoral registrations with needle points,
- a material for fixation of bite forks for use with face bows and
- as a scannable registration material.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Recommended procedures

1. Preparing the auto-mixing cartridge/R-SI-LINE[®] MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE[®] MIXING CANNULA SXN (REF MIK1093). The mixing pistol is ready for application. For direct, broaden application onto the teeth use the R-SI-LINE[®] BITE-TIPS (REF BRT1000).

2 a. Registration

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The automatically mixed R-SI-LINE[®] METAL-BITE[®] Blue is applied directly onto the occlusal areas or onto the bite-plate. The occlusion should be done within 20 s. After 45 s in the mouth, the registration has hardened and can be removed from the mouth.

2 b. Data registration (static, antagonist-registration)

The automatically mixed R-SI-LINE[®] METAL-BITE[®] Blue is applied directly onto the prepared occlusal areas. After hardening the registrate will not be removed and will be prepared mesial and distal. Then the mesial, the registrated (of the antagonist) and the distal occlusal area can be optically registered (Scan).

2 c. Data registration (dynamic, FGP-antagonist-registration)

After hardening the registrate will be prepared intraoral mesial and distal and will be reduced to a FGP-table with occlusal informations. Then fresh material will be applied onto this table and excursion movements (trusion movements) will be done. Then the dynamic registration can be scanned (literature >> www.r-dental.com).

3. Articulator

R-SI-LINE[®] METAL-BITE[®] Blue allows a visual control during fixing of the gypsum model in the articulator.

4. Storage

Do not store above 77 °F! Avoid exposure to sunlight. Keep out of the reach of children!

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Do not touch tooth surfaces, preparations or retraction cords with latex gloves.

Expiry date see label. Do not use after expiry date.

R-SI-LINE and METAL-BITE are registered trademarks of R-dental Dentalerzeugnisse GmbH. CEREC is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2023 R-dental Dentalmaterialien GmbH. All rights reserved.