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ZAKK®

CE 0482

**Hersteller:**

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

**ZAKK®**

REF ZKS2070

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REF ZKS2070

### Description and indication

ZAKK® is an auto-mixing non-eugenol temporary ZnO-cement for cementing temporary crown and bridges and for trial cementing definitive crown and bridges.

ZAKK® shows an excellent adhesion without any loss of retentions to temporary restorations and at the same time an excellent removability from prepared teeth.

The non-eugenol ZAKK® will not influence hardening of self-cure - or dual-cure resin-based composites or luting materials for definitive restorations.

ZAKK® is based on zincoxide and organic acids for an optimal marginal seal.

Especially in cases of allergies caused by eugenol ZAKK® is strongly recommended to use.

ZAKK® meets the requirements of DIN EN ISO 3107 and the ADA/ANSI-Specification No. 30.

### Recommended procedures

#### 1. Preparation of 5 ml-double-chamber syringe (KwikkMix®)

Before first use screw off the original cap and waste it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally. Then the mixing cannula is put on and locked by turning it 90° clockwise.

ZAKK® will be mixed automatically using the FANTESTIC® MIXING CANNULA MCN 2 (REF MIK1041) + FANTESTIC® INTRA-ORAL-TIP MCN 2 (REF IOT2041) of the safety syringe system.

Instead of the INTRA-ORAL-TIP MCN 2 a fine root-canal application tip is available (ROOT-CANAL-TIP, REF RCT2041), with which the material could be applied directly and efficiently.

Further on instead of of MIXING CANNULA MCN 2 + INTRA-ORAL-TIP MCN 2 a NEEDLE MIXING CANNULA (REF MIK1041N) with an integrated, bendable application needle is available.

For this type of double-chamber syringe a mixing pistol for a comfortable application is available (REF MIP1041).

#### 2. Application

Recommendation: Do not dry the stumps totally. A slight humid tooth surface will allow to remove the temporary restoration more easily.

Dispose first a small amount of ZAKK® after extruding. Introduce ZAKK® as a fine, thin layer within 90 s to the inside of the temporary crown and bridge (working time at 73 °F) and place the restoration carefully with first light, but then firm pressure onto the tooth and prepared areas.

The intraoral polymerization time is about 2-3 min. Then any excess material can be removed easily with a scaler.

After removal of the temporary restoration the polymerized cement can be easily separated from the tooth.

### Precautions to application, product life and/or storage

Avoid contact of ZAKK® with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water. If swallowed accidentally, immediately consult a physician.

Do not use ZAKK® for patients with allergic reactions against this material.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size like rice corn) should be pressed out and should be disposed of immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

Shelf life see label. Do not use after expiry date.

Do not store above 78 °F! Avoid exposure to sunlight.

The double-chamber syringe should be stored horizontally.

ZAKK and KwikkMix are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

**Caution:** For dental use only.

### Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

### Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.