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ZAKK<sup>®</sup> Implant

CE 0482

**Hersteller:**R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com**ZAKK<sup>®</sup> Implant (System 4:1)**

REF ZKS2080

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REF ZKS2080

#### Description and indication

ZAKK<sup>®</sup> Implant is an auto-mixing, semi-permanent luting cement for cementing implant abutment retained crowns and bridges.

The self-curing and light-cure (dual-cure) ZAKK<sup>®</sup> Implant is based on multifunctional methacrylic esters and contains zinc oxide.

ZAKK<sup>®</sup> Implant shows a good elastic behavior. ZAKK<sup>®</sup> Implant has good sealing properties due to a controlled slight expansion.

The non-eugenol ZAKK<sup>®</sup> Implant will not influence the setting of self-cure - or dual-cure resin-based composites or luting materials for definitive restorations.

#### Incompatibility with other materials

Do not use ZAKK<sup>®</sup> Implant in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store ZAKK<sup>®</sup> Implant in proximity of eugenol containing materials. Unpolymerized ZAKK<sup>®</sup> Implant should not come in contact with eugenol containing materials.

#### Recommended procedures

##### 1. Preparation of 5 ml-double-chamber syringe (KwikkMix<sup>®</sup>)

Before first use screw off the original cap and waste it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally. Then the mixing cannula is put on and locked by turning it 90 ° clockwise.

ZAKK<sup>®</sup> Implant will be mixed automatically using the FANTESTIC<sup>®</sup> MIXING CANNULA MPN 2 (System 4:1) (REF MIK1046).

##### 2. Application

Attention: Before cementing the abutment and the insides of the restoration must be degreased, cleaned (e. g. with alcohol) and dried.

Dispose first a small amount of ZAKK<sup>®</sup> Implant after extruding. Introduce ZAKK<sup>®</sup> Implant as a fine, thin layer within 2 min to the inside of the restoration (working time at 73 °F) and place it carefully with first light, but then with firm pressure onto the abutments.

For a fast fixation of the restoration ZAKK<sup>®</sup> Implant can be light-cured for 20 s at the marginal regions of each surface. In case of translucent materials it is also possible to light-cure ZAKK<sup>®</sup> Implant through the material (for 40 s of each surface).

ZAKK<sup>®</sup> Implant will polymerize without light-cure within 4 min. Then any excess material can be removed easily with a scaler.

##### 3. Removal for inspection

If the restoration has to be removed for inspection, this must be done gently with slight but permanent traction.

Attention: A jerky removal must be avoided!

The remaining ZAKK<sup>®</sup> Implant cement can be extracted almost in one piece. Use water spray and suction to remove any remaining residue.

The abutment and the restoration will then be cleaned and disinfected with a cotton pellet soaked in alcohol or another disinfectant. Then rinse thoroughly with water and dry.

Now the restoration can be fixed again with ZAKK<sup>®</sup> Implant, see section 2.

#### Precautions to application, product life and/or storage

Avoid contact of ZAKK<sup>®</sup> Implant with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water. If swallowed accidentally, immediately consult a physician.

Do not use ZAKK<sup>®</sup> Implant for patients with allergic reactions against acrylates.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size like rice corn) should be pressed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

Shelf life see label. Do not use after expiry date.

Do not store above 68 °F (20 °C)! Avoid exposure to sunlight.

The double-chamber syringe should be stored horizontally.

ZAKK and FANTESTIC are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

**Caution:** For dental use only.

#### Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

#### Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalmaterialien GmbH. All rights reserved.

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R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE