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ZM-Bond®

CE 0482

Hersteller:

R-dental Dentalmaterialien GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

ZM-Bond®
REF ZMF4200

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ZM-Bond®
REF ZMF4200

Description, indication and composition

ZM-Bond® is a single-component primer, which promotes a chemical adhesion between luting composites and restorative materials based on metal and zirconium dioxide. Based on an innovative MDP technology an excellent adhesion will be achieved.

ZM-Bond® has chemical groups, which on the one hand include functional groups on methacrylate basis for luting composites and on the other hand functional groups for an adhesion to indirect restorative materials and frameworks.

The luting composites on methacrylate basis (particularly FANTESTIC®-product line) can be self-curing with light-cure option that is dual-cure.

Recommended procedures**1. Preparation of indirect restorative materials or frameworks****1.1 Zirconium dioxide**

The adhesive contact areas of the restoration will be prepared according to the instructions for use of the manufacturers. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil- and waterfree air.

Attention: Do *not* use phosphoric acid on the adhesive contact areas!

1.2 Metal

The adhesive contact areas of the restoration will be sandblasted according to the instructions for use of the manufacturers, until a matt surface has been achieved. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil- and waterfree air.

Attention: Do *not* use phosphoric acid on the adhesive contact areas!

2. Application of ZM-Bond®

ZM-Bond® will be applied with a MICROBRUSH (REF MPB3981) onto the pretreated surfaces of the restorations according to section 1.1 and 1.2. The primer should be effect for 10 s and then dispense the solvent slightly with oil- and waterfree air.

3. Adhesive treatment of indirect restorations or frameworks

The luting composites can be applied to the pretreated surface in accordance with the instructions for use. The luting composites based on methacrylate (e. g. from the FANTESTIC®-product line) can be light-cure or self-curing that is dual-cure.

But use separate adhesive systems for the teeth (e. g. FANTESTIC® FLOWSIVE-product line).

Precautions to application, product life and/or storage

ZM-Bond® contains acetone. Avoid contact of ZM-Bond® with soft tissues, skin or eyes. If there is an accidental contact with eyes, flush immediately with numerous amounts of water and contact a physician. In the case of skin contact wash with numerous amounts of water. Do not use ZM-Bond® in case of allergic reactions.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F! Aceton is highly volatile. Immediately close the bottle after use. Ensure a good room ventilation. Aceton is highly flammable and irritant.

FANTESTIC and ZM-Bond are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalmaterialien GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalmaterialien GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalmaterialien GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalmaterialien GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.