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R-SI-LINE® METAL-BITE® GOLD

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

REF MBK0750 (R-SI-LINE® METAL-BITE® GOLD)

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REF MBK0750 (R-SI-LINE® METAL-BITE® GOLD)

Description and indication

R-SI-LINE® METAL-BITE® GOLD in automixing cartridges is an extremely fast, hard and precise addition curing registration material on vinyl poly-siloxane base (A-silicone) for bite-plates, too.

Characteristics

R-SI-LINE® METAL-BITE® GOLD is excellent scannable (CAD/CAM/CIM-Impression Technology). In the case of an antagonist registration procedure for example with CEREC®-cameras (SIRONA) the powderfree optical registration will lead to excellent contrast results, so that finest morphological details can be visualized with highest precision.

R-SI-LINE® METAL-BITE® GOLD is thixotropic, very suitable for cutting and contouring, inflexible and not crumbly because of its high material lattice-constant. Registrations of R-SI-LINE® METAL-BITE® GOLD are permanently dimensionally stable.

R-SI-LINE® METAL-BITE® GOLD is delivered in auto-mixing cartridges and can be directly and bubblefree syringed onto the occlusal areas and so on.

R-SI-LINE® METAL-BITE® GOLD meets the requirements of the standard DIN 13903.

Recommended procedures

1. Preparing the auto-mixing cartridge/R-SI-LINE® MIXING PISTOL N

Introduce the auto-mixing cartridge into the R-SI-LINE® MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE® MIXING CANNULA. The R-SI-LINE® MIXING PISTOL N is ready for application.

For direct, broaden application onto the teeth use the R-SI-LINE® BITE-TIPS (REF BRT1000).

2 a. Registration

Before every application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The automatically mixed R-SI-LINE® METAL-BITE® GOLD is applied directly onto the occlusal areas or onto the bite-plate. The occlusion should be done within 20 s into the desired intercuspitation (working time at 74 °F). The registrate can be taken out of the mouth after about 60 s.

2 b. Data registration (antagonist-registration) for CAD/CAM/CIM

The automatically mixed R-SI-LINE® METAL-BITE® GOLD is applied directly onto the prepared occlusal areas. After hardening the antagonist registrate will be prepared mesial and distal. Then the mesial, the registrated (of the antagonist) and the distal occlusal area can be optically registered.

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

3. Disinfection

The registrate can be disinfected with standard disinfection solutions.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately. The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Expiry date see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® METAL-BITE® GOLD with standard condensation reaction type silicone impression materials.

Do not use latex gloves. These could inhibit the hardening reaction of the A-silicone in the case of contamination.

The bite registrates can be stored over months without shrinkage.

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time:	< 20 s
Recommended time in the mouth:	≥ 40 s
Time (start mixing - withdrawal of mouth):	≥ 60 s
Dimensional change after 24 hours:	< 0,1 %
Strain in compression:	about 0,5 %
Shore-D-hardness:	about 40

R-SI-LINE and METAL-BITE are registered trademarks of R-dental Dentalerzeugnisse GmbH.

CEREC is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.